

*Your patients
will thank you
for choosing
Cristobal+
restorations!*

Cristobal+

**INDIRECT COMPOSITE
RESTORATIVE SYSTEM**



**THE PATENTED CRISTOBAL+ FORMULA FEATURES
A BARIUM BOROSILICATE GLASS COMPONENT FOR
INCREASED WEAR RESISTANCE.**

CRISTOBAL+

Restorations are proven to withstand the test of time! In independent studies, the clinical results of the historical data showed proven long-term success.

- ▶ **ESTHETIC** – excellent shade match and color stability.
- ▶ **DURABLE** – lasting natural gloss and translucency.
- ▶ **STRONG** – highly fracture resistant and shock absorbent.
- ▶ **WEAR COMPATIBILITY** – close to natural enamel.
- ▶ **BIOCOMPATIBLE** – high biocompatibility.



Ask for
CRISTOBAL+
restorations,
fabricated by:

Your full service dental laboratory!
PROTEC
DENTAL LABORATORIES LTD.



34 East 2nd Avenue
Vancouver, BC
Canada V5T 1B1
Main: 604-873-8000
Dispatch: 604-873-1212
Fax: 604-873-8527
Toll Free: 1-800-663-5488


Cristobal+®

INLAY/ONLAY PREPARATION PROTOCOL


The following preparation guidelines for restorative indications of this system will ensure the clinical success of your restorations.

INDICATIONS

- ▶ Metal-free
 - Inlays/Onlays: All teeth
 - Veneers: All teeth
 - Crowns: All teeth through 1st molar
 - Bridges: All teeth through 2nd bicuspid, one pontic bridge only
- ▶ Implants
 - All teeth

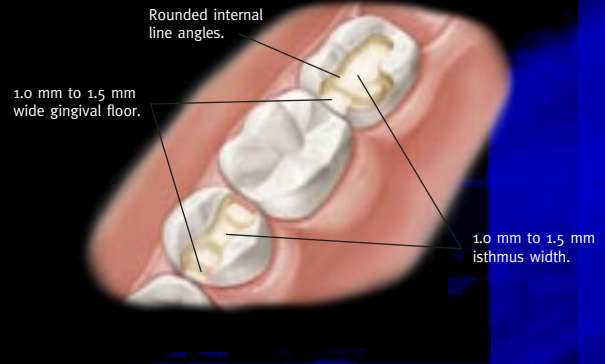

VENEER PREPARATION PROTOCOL

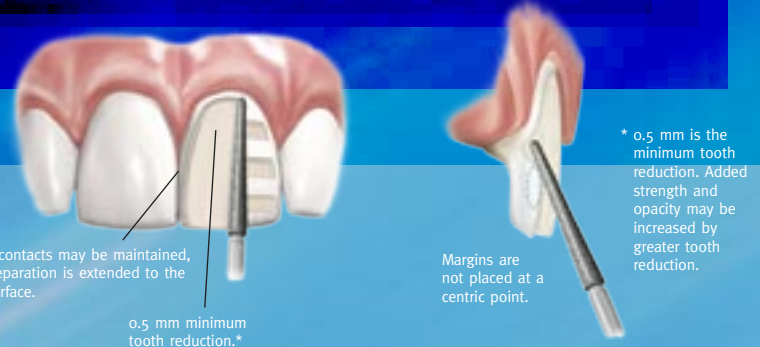


PREPARATION GUIDELINES

General specifications:

1. Use a chamfer margin around the gingival margins. Margins that extend to the occlusal or incisal must be a 90° or butt joint.
2. Avoid sharp angles.
3. Maintain an even reduction of anatomical form.
4. These are minimum reductions. Strength and opacity will be increased with more tooth reduction.

NOTE: Modifications to the above guidelines must be made with caution, using clinical judgment.

FULL CROWN PREPARATION PROTOCOL

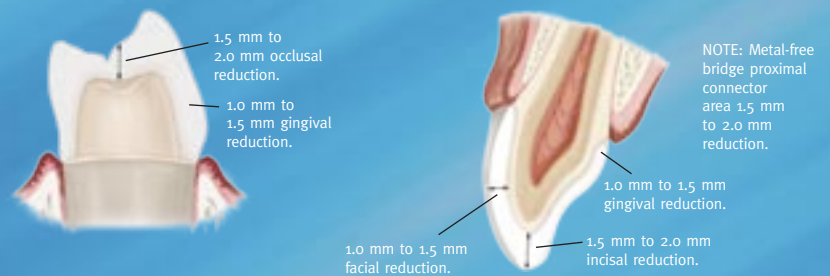

See complete directions for use for indications and contraindications.

Ask for
CRISTOBAL+
restorations,
fabricated by:



34 East 2nd Avenue
Vancouver, BC
Canada V5T 1B1
Main: 604-873-8000
Dispatch: 604-873-1212
Fax: 604-873-8527
Toll Free: 1-800-663-5488