



Care and Cleaning of Your Mouthguard



Ask for Safe-t-flex
mouthguards fabricated by:

Your full service dental laboratory!

PROTEC
DENTAL LABORATORIES LTD.



Care and Maintenance

Mouthguards, like any other sports equipment, can wear out, deteriorate, or become lost over time. Therefore, several precautions should be taken to maintain your sports guard.



**DO NOT PLACE
IN DIRECT SUNLIGHT**



**DO NOT CLEAN
IN BOILING WATER**



**DO NOT PUT
IN WASHER/DRYER**


Keep your mouth protector away from sources of heat such as the sun, washing machines, and dryers. Never attempt to clean your mouthguard by boiling it in hot water. Your guard is fabricated from a thermoplastic EVA material that will distort and melt if exposed to heat. Do not wrap your guard in a napkin or tissue as it could be mistakenly thrown out. Be careful not to leave your sports guard where it might be chewed by a pet.

Regularly check the condition of your mouthguard. Tears or perforations can irritate teeth and mouth tissues. Be sure to visit your dentist before the start of each playing season to have your oral health assessed and any problems with your mouthguard corrected.



Cleaning and Storage


BEFORE EACH USE WASH YOUR MOUTHGUARD WITH COLD WATER OR ANTIBACTERIAL MOUTHWASH

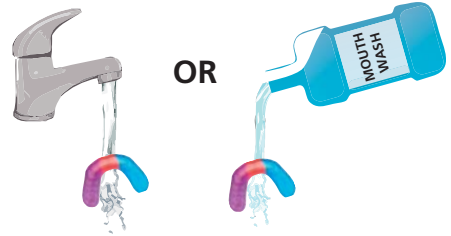

AFTER EACH USE AND BEFORE STORAGE, RINSE YOUR MOUTHGUARD WITH COLD WATER



CLEAN ALL SURFACES WITH A STIFF TOOTHBRUSH IN COLD SOAPY WATER

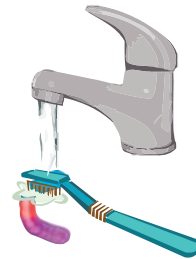

STORE IN A FIRM PERFORATED CONTAINER

