



Care and Cleaning of Your Orthodontic Appliance

Ask for orthodontic
appliances fabricated by:

Your full service dental laboratory!
PROTEC
DENTAL LABORATORIES LTD.



Care and Maintenance

Orthodontic appliances can wear out, deteriorate, or become lost over time. Therefore, several precautions should be taken to maintain your appliance.



**KEEP APPLIANCE
IN 24 HOURS A DAY**



**DO NOT CLEAN
IN BOILING WATER**



**AVOID CHEWING
ANYTHING STICKY**


Wear your orthodontic appliance 24 hours a day, unless otherwise instructed. Never attempt to clean your appliance by boiling it in hot water. Avoid chewing anything sticky such as gum or caramels. Do not wrap your appliance in a napkin or tissue as it could be mistakenly thrown out. Be careful not to leave your appliance where it might be stepped on or chewed by a pet.

Regularly check the condition of your appliance. Cracks or perforations can irritate teeth and mouth tissues or make treatment ineffective. Be sure to keep your appointments and visit your dentist regularly to have your appliance adjusted and your oral health assessed.



Cleaning and Storage

CLEAN ALL SURFACES WITH A STIFF TOOTHBRUSH AND TOOTHPASTE OR SOAPY WATER

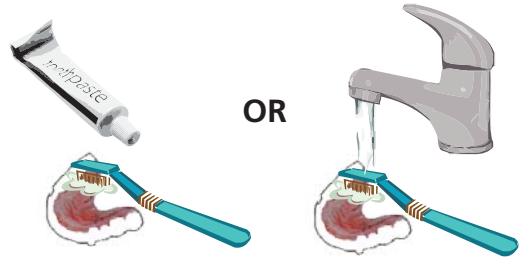

OCCASIONALLY SOAK YOUR APPLIANCE IN AN EFFERVESCENT DENTURE CLEANING SOLUTION



STORE IN A FIRM, PERFORATED CONTAINER

