

UPLOAD INSTRUCTIONS


3shape TRIOS®

- Order Creation
- Digital Scanning
- Clinical Validation
- Upload Digital File




1. *Add Protec Dental Laboratories to your system by logging into your **Communicate** account and register **Protec** as your lab.

2. Order Creation- create new case by clicking on **LAB** and choose **Protec Dental Laboratories**. Select the appropriate order form:

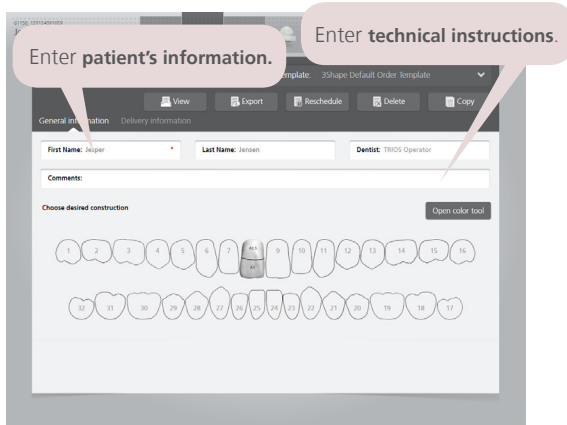


3. Digital Scanning- scan patient's maxillary and mandibular arches, and bite.

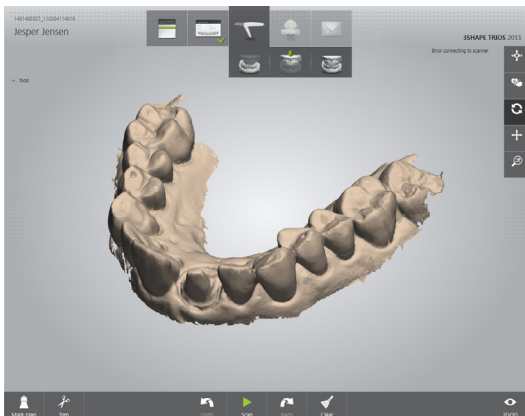


4. Clinical Validation- Review scan data in the **TRIOS Analysis** page.

Check **occlusal clearance**, view or mark margin, auto detect undercut areas.

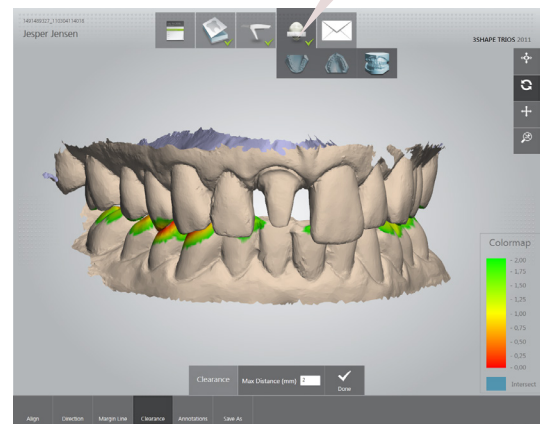

5. Upload Digital Impression- Send your digital impression to **Protec Dental Laboratories**.

Select and send case.

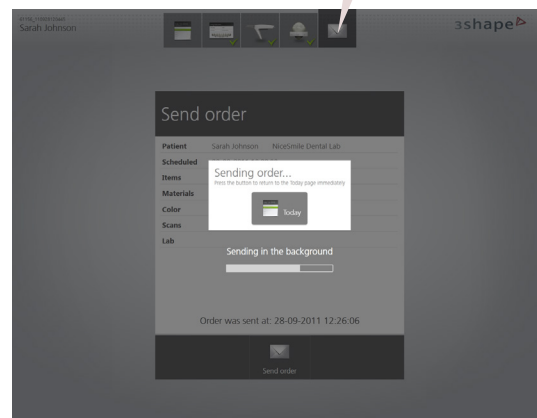

*If Protec is not in your laboratory list, please contact your TRIOS re-seller and request to add Protec.

For Restorations: communicate@protecdental.com
For Orthodontics: orthoscan@protecdental.com