

Maximize Your iTero® Scanner

with **PROTEC**

PROTEC DENTAL LABORATORIES UTILIZES STATE-OF-THE-ART NEW TECHNOLOGY AND HAS INTEGRATED DIGITAL WORKFLOW SOLUTIONS TO PRODUCE RESTORATIONS, APPLIANCES, AND MODELS FROM YOUR DIGITAL IMPRESSIONS.



The iTero Intraoral Scanner is synonymous with high precision. Now, the iTero® Element™ scanner is engineered to deliver everything that doctors love about iTero scanners in a compact footprint with even bigger capabilities. The iTero Element scanner is designed to put the latest technology at your fingertips and make intraoral scanning fun.

PROVEN IMAGING PRECISION

- CERTIFIED CONNECTIVITY WORKFLOWS
- ADVANCED WAND FEATURES
- REAL-TIME VISUALIZATION
- 6,000 FRAMES PER SECOND
- HIGH-DEFINITION MULTI-TOUCH DISPLAY
- 3D COLOUR SCANNING
- COMPACT FOOTPRINT DESIGN

PROTEC
DENTAL LABORATORIES LTD.

Crown & Bridge • Dentures • Orthodontics



YOUR DIGITAL TECHNOLOGY PARTNER LABORATORY!

UPLOAD INSTRUCTIONS




- Call iTero Customer Support
- Order Creation
- Scan Digital Impression
- Send case




1. *Add **Protac Dental Laboratories** to your system by calling **iTero Customer Support line at 1-800-577-8767.**

2. Click on the **Rx icon** in the top-left of your interface to open the **Treatment Information** window

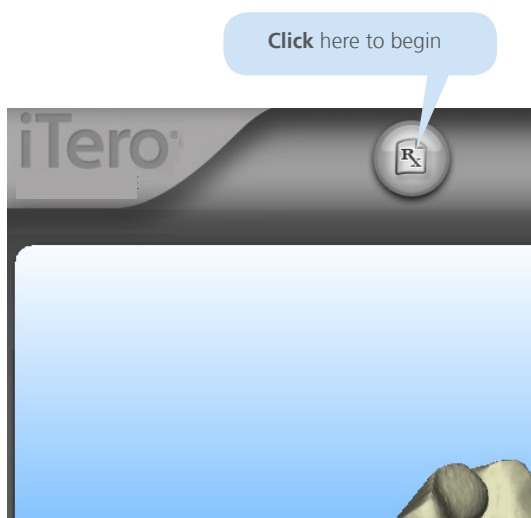

4. Select **Protac Dental Lab (for Crown & Bridge, Implants, or Partials),** or **Protac Ortho Lab (for Orthodontics, Aligners, or Splints)**

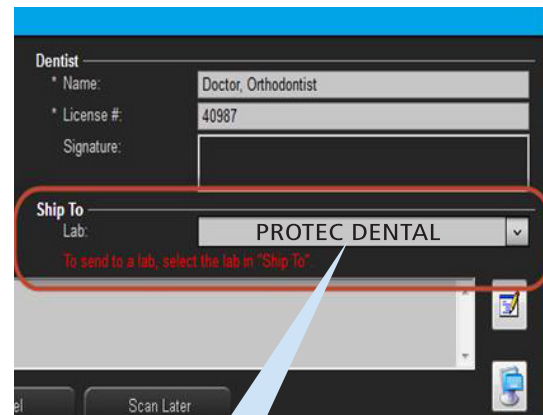

5. Scan digital impression



3. Enter **Patient information** and select **iRecord** under **Scanning Order**



6. Send case to **Protac Dental Laboratories!**

*If Protac is not in your laboratory list, please contact **iTero Customer Support.** Indicate the lab ID below:

Restorative: Protac Dental Lab, ID 170
Orthodontics Protac Ortho Lab, ID 12661